

■ PRODUCT DATASHEET

C2 Standard Urine Bag, 2L, NRV, CleanPak™

Brand	: CURAS®
Category	: Urology
Range	: Drainage Bags
Product Group	: C2 Standard Bags
REF	: 20421, 20424, 20426



PRODUCT DESCRIPTION

Non-sterile, 2L, disposable standard urine drainage bags, with non-return valve. Each urine bag is packed individually in CleanPak™, inner bag to enhance the hygiene performance, convenience of usage and distribution.

INTENDED USE

Curas standard urine bags are used for short-duration collection of urine.

DURATION OF USE

The bag is for short-duration used. Recommended for the duration of 1 -2 days, under normal use. Properties of the product is subject to user condition, e.g. frequency in draining, medical condition, mobility of patient and other product-stress factor. For Hygienic reason, frequent cleaning of bag is recommended. New bag is required whenever the catheter is change.

Duration of Use

- Short Duration (1 – 2 days)
 Medium Duration
 Long Duration

FEATURES & BENEFITS

- Cost-effective solution for everyday use.
- Hygienic alternative to the bulk-pack standard bag for short-duration use.
- Packed individually in CleanPak™ (inner bag) for increased user safety and convenience.
- Clear ml-graduation markings allow for easy reading of the amount of urine collected.
- Measures as low as 25ml, blower than other standard bags in the market (e.g. 100ml).
- Ridged universal connector fits all common catheters.
- Flexible tube with outer diameter of 6.4mm, comes in different lengths to suit user needs.
- Non-return valve (NRV) prevents reflux into the bladder, reducing the risk of ascending infection.
- Reinforced eyelets for safer use with disposable hanger (e.g. REF 20421, REF 20424), REF 20426.
- Double-sealed film for extra strength.
- 'T'-valve/cross-valve permits fast and efficient emptying and one-hand drainage operation by user/nurse. (e.g. REF 20421, REF 20424)
- Strong push-pull valve (e.g. REF 20426) provides extra strong feature to help fast and efficient urine emptying.
- Medium bag, WardPak™ allows easy dispensing at ward level.

PRODUCT SPECIFICATIONS

REF	Capacity	Tube Length	Bottom Outlet	Packing
20421	2L	90cm	Cross Valve	CleanPak™
20424	2L	110cm	Cross Valve	CleanPak™
20426	2L	90cm	Strong Push-Pull Valve	CleanPak™

Composition

- Bag and drainage tube made of PVC material.
- Drainage outlet and protective caps made of PE material.
- All components are 100% latex-free.
- Material contains DEHP phthalates.

Country of Origin

- China

Sterilization

- Non-sterile

Shelf Life

- 8 years

Selected Standards & Guidelines

- MDD 93/42/EEC (CE Marking, Class I), as amended by Directive 2007/47/EC
- EN 980:2008
- EN 1041:2008
- EN ISO 13485:2016
- EN ISO 14971:2007

Recommended Product Storage

- Store at temperature between 10°C-25°C / 50°F-77°F and avoid exposure to elevated temperatures.
- Do not expose product to radiation, sunlight and UV light, as it will change the properties of the product.

Proposed Disposal Method

- Dispose of used product into clinical waste bag or according to clinical waste protocol, e.g., incineration.

Product Symbols



PACKAGING INFO

REF	EAN-13 Barcode	CleanPak™	WardPak™	Carton	Gross Weight
20421	5051400204210	1 pc / pack	25 pcs / bag	250pcs / carton	~10 kg
20424	5051400204241	1 pc / pack	25 pcs / bag	250pcs / carton	~12 kg
20426	5051400204265	1 pc / pack	25 pcs / bag	250pcs / carton	~10 kg



PF-723/01/170718/00

Components / Material Properties			Recommendations on Waste Handling			
Packaging	Material	Contains PVC	Incinerate	Landfill	Recycle	Reuse
CleanPak™	PE	--	√	(√)		
WardPak™	PE	--	√	(√)		
Outer Carton	Corrugated Carton	--	(√)	(√)	√	
Transport Pallet	Wood	--			(√)	√

√ = Highly recommended (√) = Recommended

ADDITIONAL INFO

- HS Code: 9018908400
- Documentation available upon request. Kindly contact: info@curas.com